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The push rod for the Trets Trike is suited as a holding / pushing aid and to control the course of your child. The push rod is height adjustable and easy to mount.

You will need following tools:

- Allen key (size 4 and 5)
- wrench (size 8)
- pointed object (for example a scissors)

Assembly

1. Loosen the three belts on the back of the wheel cover. Loosen the seat cover from the velcro straps on the seat frame. Now you can slide the seat cover upwards above the seat frame.
2. Set the W-4 32 pipe clamps (1) with the rubber element on the seat frame. Tighten the support plate (2) with two M6x35 bolts (9) at the pipe clamps by using a 5mm allen key. Place 4 spacers (12) at both sides.
3. Fit the seat cover. Prick with a pointed object two holes into the seat cover to assemble the M5x50 bolts (10).
4. Set the small pipe clamps (3) around the cross bar. Use the M5x50 bolts to mount the upper collar (4) at the support plate and the M5x45 screws for the other collar (9). Place the sleeve (5) between the collars and tighten the bolts by using a 4mm Allen key.
5. Place the alu clamp (6) on the sleeve. Slide the push rod (7) into the sleeve. Set the desired height and tighten the clamp.

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