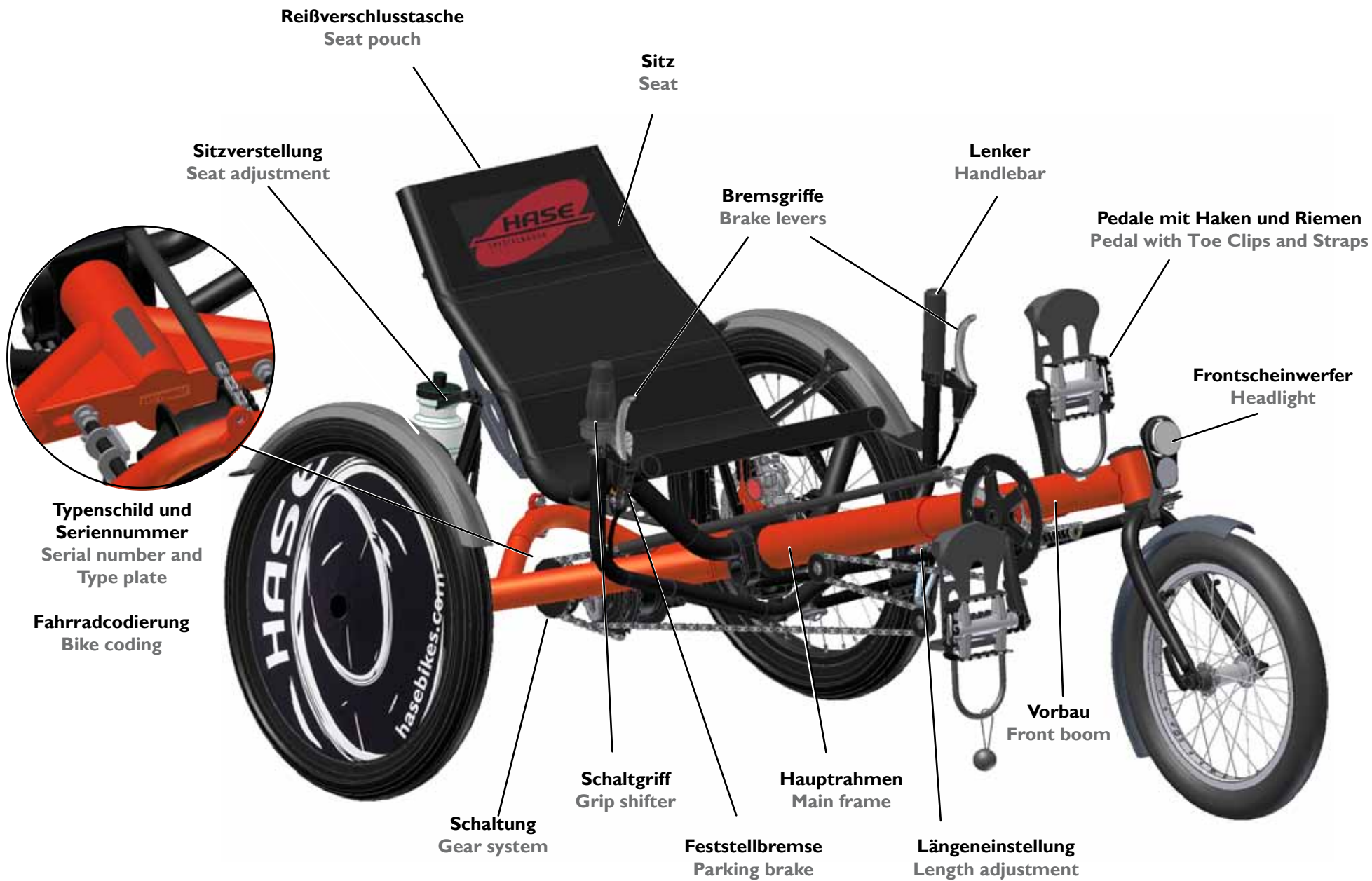
***TRIX***

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**User's Manual**

## Inhalt

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## Contents

<b>Trix Accessories</b> .....	<b>2</b>	<b>Maintenance and repair</b> .....	<b>21</b>
<b>Description</b> .....	<b>3</b>	Maintenance intervals .....	21
What is the Trix? .....	3	Checking wheels for lateral/radial true .....	22
Intended use .....	3	Checking tires .....	22
Indications for use .....	3	Changing tires or repairing a flat .....	23
Contraindications .....	3	Adjusting the fenders .....	23
Adaptability to individual needs .....	3	Adjusting the dynamo (optional) .....	23
Multiple users .....	4	Checking disc brakes .....	24
Protection against environmental influences .....	4	Checking and adjusting the internal gear .....	24
Cleaning and disinfection .....	4	Checking the chain .....	25
Applicable standards and regulations .....	4	Cleaning and oiling the chain .....	25
<b>Safety information</b> .....	<b>5</b>	Cleaning and disinfecting your Trix .....	26
Read User's Manual .....	5	Checking the chain tube .....	26
Before your first ride .....	5	<b>Installing accessories</b> .....	<b>27</b>
Observing traffic regulations .....	5	Installing the universal coupling .....	27
Avoiding muscle strain .....	5	For use with a second Trix in tandem .....	28
Tipping and overturning .....	6	For use with a BOB trailer .....	28
Braking .....	6	For use with a Weber Monoporter .....	29
Suitable clothing and helmet .....	6	For use with a Weber coupling and straight drawbar .....	29
Risk of injury from rotating parts .....	6	For use with a Weber coupling and curved drawbar .....	30
Crushing hazard .....	6	Attaching the headrest .....	31
Risk of flammability .....	7	<b>Long-term storage and disposal</b> .....	<b>50</b>
Pedals .....	7	Preparing the Trix for long-term storage, and for use after long-term storage .....	50
Use of a seat pad .....	7	Preparing the Trix for disposal .....	50
Being visible for others .....	7	<b>Conditions of warranty and liability</b> .....	<b>51</b>
Riding at night .....	7		
Technical condition .....	7		
Replacement parts and accessories .....	7		
Pulling a bicycle trailer .....	8		
Loading the Trix .....	8		
Serial Number and Bike Coding .....	8		
<b>Adjusting your Trix</b> .....	<b>9</b>		
Checking the seat position .....	9		
Adjusting the frame length .....	9		
Adjusting the steering .....	11		
Adjusting the handlebar width / angle .....	12		
Adjusting the seat angle .....	12		
<b>Riding your Trix</b> .....	<b>13</b>		
Attaching the flag .....	13		
Correct tire pressure and technical condition .....	13		
Mounting and dismounting .....	13		
Shifting gears with the twist shifter .....	14		
Braking .....	14		
Transporting luggage .....	15		
Seat pouch and fender-stay pouches .....	15		
Pulling a trailer or a second Trix in tandem .....	15		
Parking brake .....	16		
Parking the Trix upright .....	16		
Transporting the Trix .....	17		
Packing the Trix .....	18		
Transporting the Trix on a car rack .....	18		
Removing and fitting the front boom .....	19		
Checking and adjusting the alignment of the front boom .....	19		

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**Trix with quick-adjust frame**  
 The length of the quick-adjust frame can be adjusted without changing the chain length.

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Hydraulische  
 Scheibenbremsen (Option)  
 Hydraulic disc brakes  
 (optional)

Armlehne  
 Armrest

Differential (Option)  
 Differential (optional)

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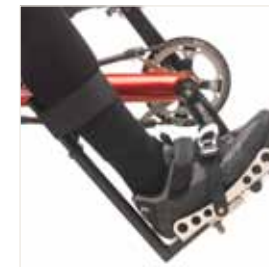
## Description

### What is the Trix?

The Trix is a recumbent children's trike and an approved medical device as per the European Directive 93/42/EEC (Medical Device Directive). The trike is driven by leg power via the pedals, which are specially designed to prevent the feet from slipping out when riding. The transmission is a fully enclosed 8-speed hub gear system, which drives one or, optionally, both rear wheels. The padded seat has been specially designed and constructed with flexible materials to cushion the ride and offer maximum comfort. The Trix has a reliable brake system with two independently operated disc brakes.

### Intended use

The Trix is intended for use on paved roads and paths. It can also be adapted for adult use. The required technical adjustments, however, must be performed by an authorized retailer.

The Trix is not approved for off-road use or racing. It should not be ridden in traffic until it has been equipped with the accessories required by the applicable traffic regulations of the country in which it is being used.

### Indications for use

The Trix is designed for children with neuromuscular disorders, such as cerebral palsy or muscular dystrophy, who – owing to special physical conditions – are unable to ride standard children's bicycles or vehicles, even with training wheels. Other indications include coordination disorders, balance disorders and restricted movement associated with spasticity, paraplegia, dysmelia, amputations, etc.

### Contraindications

The Trix is not suitable for children with visual impairment.

### Adaptability to individual needs

The Trix can be equipped to accommodate riders with various physical impairments. The following are just a few examples of available accessories: single-hand controls, crank shorteners, special pedals with or without calf support, pedal pendulums, lap and shoulder belts, and holders for walking sticks.

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### **Multiple users**

Trix trikes that have been used in accordance with their intended use can be re-used by a new rider after the trike has undergone an inspection by an authorized retailer.

### **Protection against environmental influences**

The Trix is constructed using special materials that protect it against corrosion: powder-coated aluminum frame parts, plastics, and stainless steel.

### **Cleaning and disinfection**

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. For more information, see the section “Cleaning and disinfecting your Trix” on page 26.

### **Applicable standards and regulations**

The Trix has been designed and manufactured in compliance with the following regulations and standards:

European Directive 93/42/EEC (Medical Device Directive)

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## Safety information

### Read User's Manual

Read this User's Manual carefully and take the time to familiarize yourself with the construction and function of the trike in practice. Share this information with the child who will be riding the trike. Be sure that the child has understood all safety-related functions.

### Before your first ride...

Your authorized retailer has assembled your new Trix and adjusted it to your size requirements. Please refer to the section "Adjusting your Trix" on page 9 if you need to make adjustments for another rider. Before setting out on your first trip, please take some time to familiarize yourself with the Trix. Due to its unique construction, there are significant differences in handling in comparison with a standard bicycle. Accompany the child on a test ride in an open area away from traffic, and allow the child to practice riding until (s)he has mastered all important riding skills.

### Observing traffic regulations

The Trix comes with a lighting system and all equipment required by the German road traffic regulations (StVZO). Any changes, adjustments, or repairs must be made by a qualified bike mechanic. Traffic regulations are different in every country. It is important to ensure that the trike is properly equipped for traffic in the respective country prior to use.

In Germany, children under the age of 8 must ride on sidewalks accompanied by an adult, even if the respective road has a bicycle lane. Children who are at least 10 years of age can ride in traffic accompanied by an adult. Between the ages of eight and ten, children can ride on sidewalks or in bicycle lanes. Be sure to follow the applicable regulations in the country of use, and observe the respective rules for adult supervision.

### Avoiding muscle strain

In a recumbent position, the rider uses different muscle groups than with a standard bicycle. Riders with little or no experience on recumbent bikes/trikes should begin with short, easy trips in order to avoid excessive muscle strain.

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### **Tipping and overturning**

Recumbent tricycles have a low center of gravity. This allows for cornering at higher speeds. The centrifugal force felt when turning at high speeds, however, can cause the trike to skid or even tip over, which could cause injury to the rider or others. Avoid making hectic steering movements or braking in curves. This increases the trike's tendency to tip.

Be careful when stopping on inclines. The Trix could roll backwards and, if brakes are applied abruptly, even overturn. Always wear a helmet for your own safety.

### **Braking**

The disc brakes of the Trix are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before your first trip in public traffic, be sure to take the time to familiarize yourself with the braking strength by trying a few braking trials in an open area away from road vehicles.

### **Suitable clothing and helmet**

Do not wear loose clothing that could get caught in the wheels or gears. Beware of scarves and clothing strings/cords that hang down. Always wear a helmet in order to avoid head injury in the case of an accident.

### **Risk of injury from rotating parts**

The wheels of the Trix are equipped with spoke guards that eliminate the risk of spoke-related injury. However, the trike has other rotating parts that pose hazards. The Trix rider should be advised never to touch the disc brakes or gears, which could cause severe injuries. Never engage/disengage the dynamo when the trike is in motion. Be sure to discuss these dangers with the Trix rider.

### **Crushing hazard**

The Trix has been designed with numerous adjustment sites to make it adaptable to many different riders. Therefore, it was not possible to completely avoid joints that pose a crushing hazard. To prevent injury, be careful not to get your hands or feet caught or pinched in the trike's parts when riding or adjusting the trike. Be sure to discuss these dangers with the Trix rider.

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## **Risk of flammability**

The seat cover is made of a flammable material. If exposed to an open flame, such as a lighter, the seat cover could catch fire, which could cause severe burns to the Trix rider. Therefore, avoid any use of fire near the seat cover. Be sure to Risk of flammability fire, which could cause severe burns to the Trix rider. Therefore, avoid any use of fire near the seat cover. Be sure to Risk of flammability discuss these dangers with the Trix rider.

## **Pedals**

In a recumbent position, it is especially easy for the rider's feet to slip from the pedals. Therefore, be sure to use the mounted pedals with toe clips and to secure the feet using the attached straps to prevent them from slipping out of the toe clips.

## **Use of a seat pad**

Please keep in mind that using a seat pad to raise the sitting position can change the center of gravity and thus also the handling of the Trix. Therefore, before riding in traffic, the rider should take time to familiarize herself with the handling of the Trix with the seat pad in an open area away from traffic.

## **Being visible for others**

The Trix is lower than an upright bicycle. This may make it more difficult for other road vehicles to see you. Always ride with the flag supplied with your trike. It attracts the attention of other road users.

## **Riding at night**

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn.

## **Technical condition**

The Trix should only be ridden in perfect technical condition. Check for damage before each trip. Take the Trix to your bike shop for regular maintenance. The recommended maintenance intervals can be found in the section "Maintenance intervals" on page 21.

## **Replacement parts and accessories**

Use only the replacement parts and accessories recommended by HASE for the Trix. If other parts are used, the warranty will lose its validity. HASE is not liable for any resulting damage. Consult your bike shop for more information.

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### **Pulling a bicycle trailer**

Accessories are available for attaching bike trailers to the Trix. If you live in Germany, however, keep in mind that, according to German road traffic regulations (StVZO), only riders who are at least 16 years of age are allowed to pull a bike trailer.

### **Loading the Trix**

Be sure not to exceed the maximum permissible combined weight for rider and luggage: 120 kg (265 pounds). All luggage should be secured using the mount behind the seat or a low-rider rack (accessory). Always ensure that the load is distributed evenly.

### **Serial Number and Bike Coding**

The serial number can be found on the end of the main frame (see illustration on the inside cover of this manual). This is the only place where bike coding can be engraved. Bike coding helps authorities locate the bicycle owner in the case of theft or loss.

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## Adjusting your Trix

The proper adjustment of the Trix is important for riding comfort and performance. The Trix can be adjusted to the dimensions of the riders from approx. 1.20 to 1.65 m (4' to 5'5") in height, or up to 2 m (6'7") with change of crank. Models with a quick-adjust frame can be adjusted to any length within the maximum range of 25 cm. Larger adjustments require a modification of the quick-adjust assembly and chain length.

### Checking the seat position

Have the child sit in the seat of the Trix and place her feet on the pedals with the balls of her feet on the pedal axles. When the crank arms are lined up with the front boom, one leg should be almost straight (see above photo) and in a comfortable position. If the leg cannot be straightened or the foot cannot fully reach the pedal, then the length must be readjusted.

### Adjusting the frame length

1. Loosen the two Allen bolts between the front boom and the main frame (1) using a 6 mm Allen key. Position yourself on the right side of the Trix (when facing in the direction of travel).
2. Grasp the front wheel in one hand and the handlebar in the other.
3. Pull the wheel and the handlebar toward one another in order to decrease the length or push them away from one another to increase the length.

#### Note:

If you want to adjust the length by more than approx. 15 to 20 cm (6 to 8 in), you may have to adjust the length of the steering rod as an intermediate step (see Adjusting the steering, page 11) and then continue with the frame length adjustment.

4. If the length has been adjusted beyond the 25cm maximum range, the quick-adjust assembly must also be readjusted.  
**Shortening the frame:**  
Loosen the front clamp (1) and slide the tube forward (e.g., by 10 cm). Retighten the clamp and shorten the chain by an amount equal to twice the distance by which the assembly was adjusted (in this example, 20 cm).

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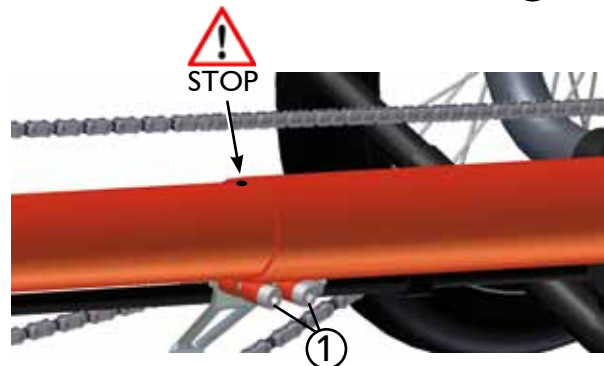
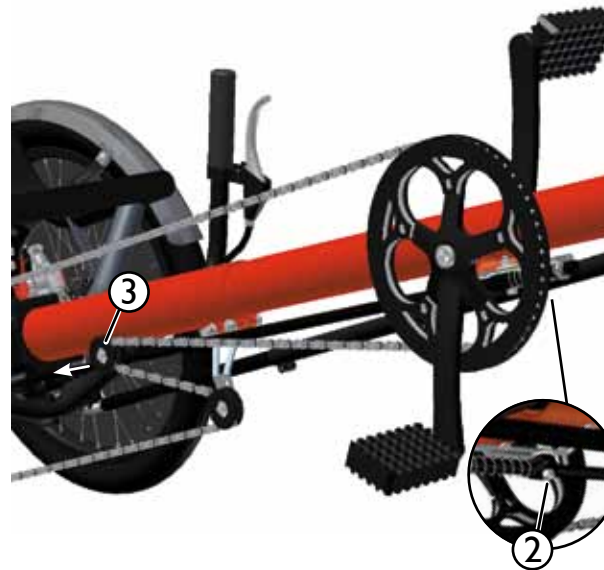
**Note:**  
If it is not possible to push the front boom further into the frame, check whether the 12mm tube of the quick-adjust mechanism is already coming in contact with the clamp.

Lengthening the frame:

Loosen the clamp (2) of the rear idler pulley and slide it 8 cm toward the tube end. Retighten the clamp, and lengthen the chain by 16 cm.

**Caution!**  
The front boom cannot be extended past the minimum insertion mark, which is a small hole located on the top of the boom. This hole should not be visible. If the hole is visible, the front boom has been extended past its minimum insertion length. Riding with an over-extended boom could cause accidents or damage to the frame.

5. Check the seat position. If further adjustment is required, repeat steps 2 to 4.
6. The bottom bracket axle must be horizontal. See page 19.
7. Tighten the two Allen bolts (1) (above photo) between the front boom and the main frame using a 6mm Allen key and a torque of 12 Nm (8.9 ft-lbs).

**Caution!**  
It is absolutely necessary to tighten these bolts with a torque of 12 Nm (8.9 ft-lbs). Riding with loose bolts poses a high risk of accident and damage to the frame.

**Note:**  
Longer crank arms are also available (accessories) for taller riders. These allow the Trix to grow with the child into adulthood. When converting the Trix for adult use, the chain may also have to be lengthened.

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### Adjusting the steering

The movements of the handlebars are conveyed to the front wheel by means of a steering rod (1). The steering rod consists of two tubes, the smaller of which fits inside of the larger and can be moved to adjust the length. Their position is fixed by means of a clamping bolt. The steering rod must be readjusted if the length of the Trix has been changed.

1. Loosen the Allen bolt (2) on the steering rod (left side of the Trix) with a 5mm Allen key. Position the front wheel so that it is pointing forward and hold it in this position.
2. Position the handlebar so that it is perpendicular to the main frame.
3. Tighten the Allen bolt (2) on the steering rod with a 5mm Allen key.
4. Check that the handlebar has full freedom of movement by moving it as far as it will go to the left and to the right.

#### **Caution!**

Any restrictions in handlebar movement could lead to accidents with possible injuries.

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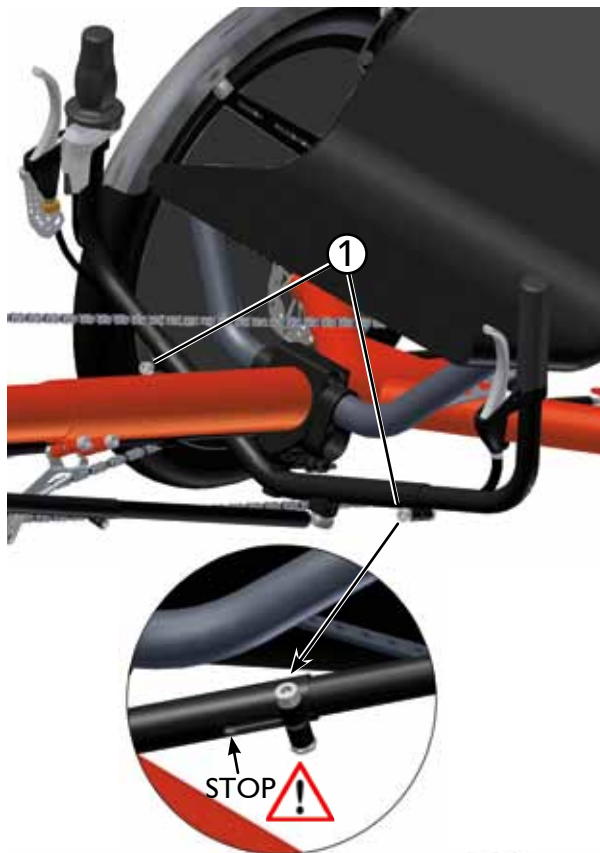
### Adjusting the handlebar width / angle

You can adjust the width and angle of the handlebar to fit your shoulder width and arm length.

1. Loosen the bolts (1) on each side of the handlebar using a 6mm Allen key.
2. Pull out the handlebar grips in order to increase the width, or push the grips further in to decrease the width.

#### **Caution!**

The ends of the grips should not be visible through the slots in the center section of the handlebar. This means they have been pulled out past their minimum insertion length. Major risk of accident! If too much pressure is placed on the grips, the center section could break.

Make sure that the handlebar grips are not pulled out so far that they bump against the wheels or fenders. Major risk of accident! The steering could lock in the handlebar's maximum left or right position.

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### Adjusting the seat angle

1. Loosen the two quick releases (1) of the seat stays.
2. Adjust the seat angle to the desired position.
3. Retighten the two quick releases (1).
4. Tighten the two lower bolts of the seat stays (2) using a 5mm Allen key.

#### **Note:**

For Trix models with a lower seat position, the upper slots can also be used for fastening the seat stays.

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### Attaching the flag

#### **Caution!**

The Trix is lower than an upright bicycle. This may make it more difficult for other road vehicles to see you. Use the included flag at all times.

Push the pole of the Trix flag through the loop (1) on the left side of the seat cover (or on the right side for driving on the left) and brace the end by wedging it in the triangular opening of the seat stay.

### Correct tire pressure and technical condition

The Trix should only be ridden with the correct tire pressure. The tire pressure range is specified on the tire sidewalls. The two rear wheels are equipped with spoke guards. Therefore, in order to inflate the tires, you must first pull apart the Velcro opening. The Trix should not be ridden unless it is in perfect working order.

#### **Caution!**

Tire pressure affects the handling and safety of the Trix. Therefore, always ride with the tire pressure indicated on the side of the tire.

### Mounting and dismounting

The parking brake prevents the Trix from rolling when mounting or dismounting.

1. Engage the parking brake by pulling down the parking brake lever (3) on the right handlebar grip. For hydraulic brakes (optional), compress the right brake lever and wrap the Velcro strap around the brake lever to hold it in an engaged position.
2. The parking brake is now engaged.
3. Sit down on the seat.
4. Place one foot on the respective pedal, sliding it into the toe clip (4), and pull the elastic strap (2) around the heel.
5. Place the other foot on the other pedal, sliding it into the toe clip, and pull the elastic strap around the heel.
6. Fasten the lap and shoulder belts (accessories) if available.
7. Disengage the parking brake by moving the lever back to its original position or – for hydraulic brakes – removing the Velcro strap.

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## Shifting

Shifting gears allows you to adjust pedaling strength to the terrain and desired speed. The force required per pedal revolution can be decreased by selecting the appropriate gear. In this way, it is much easier to ride up a steep hill in a lower gear than in a high gear. In a lower gear, however, you are slower and have to pedal faster. High gears are best suited for riding at higher speeds on flat terrain or riding downhill. Select a gear that allows you to pedal at more than 70 pedal revolutions per minute. This cadence is optimal for producing sustained power and preventing excessive strain on the knee joints.

Trix is equipped with an 8-speed hub gear with twist shifter. Hub gears allow you to shift even when the pedals are not in motion.

**Caution!**  
Please note the information provided in the enclosed manual from the gear system manufacturer.

**Caution!**  
Before riding the Trix on public roads, be sure to familiarize yourself with gear shifting in an open area away from traffic. Shifting gears takes practice, which could distract you from navigating road traffic.

## Shifting gears with the twist shifter

Turn the twist shifter (1) on the right handlebar clockwise to shift to a higher gear for faster riding or counterclockwise to shift to a lower gear for slower riding.

## Braking

In its standard version, the Trix is equipped with mechanical disc brakes. Hydraulic disc brakes are also available.

The brake levers (2) are mounted on the left and right grips of the handlebar. The left wheel brake is controlled with the left brake lever, and the right wheel brake with the right brake lever.

**Caution!**  
The disc brakes of the Trix are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before riding the Trix on public roads, be sure to familiarize yourself with the braking force by trying a few braking trials in an open area away from traffic.

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## Transporting luggage

Special mounts (1) for a bicycle bag are located behind the seat. These mounts are suitable for various models of bags. We recommend the appropriate Ortlieb bag for recumbents. Other hook-mounted bicycle bags, such as VauDe bags, can also be used. However, please make sure that these bags cannot come in contact with the rear sprocket.

### Note:

When transporting larger quantities of luggage, be sure to distribute the load evenly. An unbalanced load can cause difficulties in road handling.

## Seat pouch and fender-stay pouches

A small zippered pouch (2) is built in to the seat back. This pouch can be used for storing small objects. One or two additional pouches (optional) can be mounted on the fender stays (3) for transporting small articles such as keys, sunglasses, etc.

### Note:

These pouches are also perfect for keeping our Hase postcards on hand for interested passers-by. This saves you from having to make lengthy explanations. The postcards can be reordered from our office at any time.

## Pulling a trailer or a second Trix in tandem

With the universal coupling (4) (accessory), the Trix can be used in combination with a second Trix or Kettwiesel as a tandem trike. You can also attach a trailer with a Weber coupling, a BOB trailer, or a Weber Monoporter. If you live in Germany, however, keep in mind that, according to German road traffic regulations (StVZO), only riders who are at least 16 years of age are allowed to pull a bike trailer.

The Trix can also be pulled as a trailer behind a Kettwiesel.

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### Parking brake

The Trix is equipped with a parking brake to prevent it from rolling when parked.

Trikes with mechanical disc brakes have a lever (1) on the right handlebar grip.

1. Move the lever forward.
2. The brake is now engaged.
3. Move the lever back. The brake is disengaged.

For tricycles with hydraulic disc brakes, a Velcro strap (2) is attached to the left handlebar grip.

1. Pull the brake lever.
2. Wrap the Velcro strap (2) around the brake lever and attach the strap to the grip.
3. The brake is now engaged.
4. Remove the Velcro strap (2). The brake is disengaged.

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### Parking the Trix upright

The Trix can be parked upright with the front wheel raised. In this position, it requires less space for parking than a standard bicycle. A plastic parking stand is mounted behind the seat to protect the Trix from damage in this position. To bring the Trix into the upright parking position, simply lift the front wheel.

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## Transporting the Trix

The Trix can be adjusted to its most compact state for transportation by car, train, or other vehicle (see section “Adjusting the frame length” on page 9). The front boom can be fully inserted or removed, and the seat can be folded forward or backward.

1. Mark (for example, with a permanent marker) the position of the front boom in the main frame on the bottom of the tube. This makes it easier to reassemble the trike with its original dimensions.
2. Adjust the frame to its minimal length. For more information, see section “Adjusting the frame length” on page 9.
3. Loosen the quick release on the fork and remove the front wheel.

### Caution!

Be careful not to damage the front fender.

4. Loosen and remove the two quick releases (1) of the seat stays.
5. Screw the two quick releases into the seat bracket slots to avoid losing them.
6. Fold the left seat stay back and the right seat stay forward. The two lower bolts (2) of the seat stays may have to be loosened slightly beforehand.

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or

- Unfasten the seat cover beneath the seat and fold the seat forward.
- Now the Trix is ready for transport.

**Caution!**

Be sure to protect all sensitive parts, such as the disc brakes and rear derailleur, from damage.

If you are planning to take the Trix on an airplane or have it shipped by a forwarding company, we recommend packaging it in a cardboard box for protection.

**Packing the Trix**

Use a cardboard box with internal measurements of at least 137 x 86 x 55 cm (54 x 34 x 22 in). Adjust the frame to its minimal length, and remove the front wheel. Place the Trix in the box and secure it with packing material to prevent movement. Be sure to protect all sensitive parts from damage. If necessary, remove the front fender.

The packing dimensions can be further reduced by removing the two rear wheels and fenders. See page 35.

**Transporting the Trix on a car rack**

The Trix can be transported with a standard roof or rear-mounted bicycle carrier. The rack must be equipped for three bicycles and adjusted so that each Trix wheel can be anchored on a separate wheel tray. Secure all wheels and the frame using load straps. Check the mounting to be sure that the Trix cannot come loose during the trip. Remove all parts that could fall off in transport.

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## Removing and fitting the front boom

The front boom can be removed from the main frame to further reduce the frame length.

1. Remove the chain from the chainring.
2. Loosen the two Allen bolts (1) between the front boom and the main frame using a 6mm Allen key.
3. Loosen the Allen bolt (2) on the steering rod (3) (on the left side of the Trix) using a 5mm Allen key; then pull the front boom out of the main frame and the inner tube of the steering rod assembly out of the outer tube.
4. Be sure to protect the front boom and steering rod assembly from damage.
5. To fit the front boom, follow the above instructions in reverse order. Insert the front boom into the main frame, ensuring that the groove plate (4) shown on the following page fits inside the groove in the front boom.
6. Align the front boom (see following section); then adjust the steering (see page 11).

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## Checking and adjusting the alignment of the front boom

The front boom must be aligned in the main frame in such a way that the plane of the front wheel is perpendicular (at a 90° angle) to the ground. To ensure that the front boom does not rotate out of alignment (e.g., when adjusting the frame length), it is braced by a small groove plate (4) (next page) inside the frame tube that fits inside a groove in the front boom. This plate has an eccentric screw (6) (next page) that, when turned, pushes the plate to the side in order to adjust the rotational angle of the front boom. The position of this plate rarely requires readjustment. However, if the front wheel is not positioned perpendicular to the ground, adjust the groove plate as follows:

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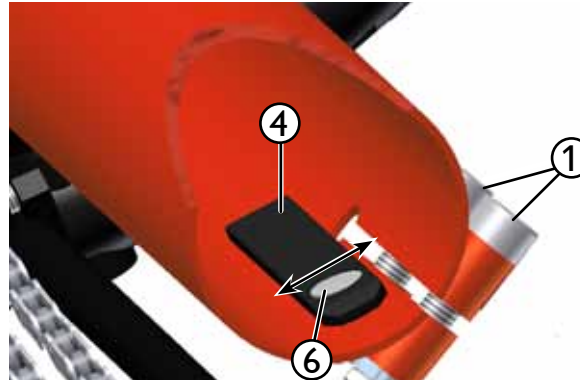
1. If the front boom has been removed from the main frame (e.g., for transport), insert it back into the frame tube, ensuring that the groove plate (4) slides inside the groove of the front boom.
2. Check the alignment of the front boom. It must be positioned in such a way that the plane of the front wheel is perpendicular (at a 90° angle) to the ground.
3. If the two Allen bolts (1) between the main frame and the front boom have not been loosened, then loosen them using a 6mm Allen key.
4. Loosen the locknut (5) with a 13mm wrench and turn the eccentric bolt (6) in the groove plate (4) using a 4mm Allen key until the plane of the front wheel is positioned perpendicular to the ground.
5. Then, while bracing the eccentric bolt (6) with a 4mm Allen key, retighten the locknut (5) using a 13mm wrench.
6. Finally, tighten the two Allen bolts (1) between the front boom and the main frame using a 6mm Allen key and a torque of 12 Nm (8.9 ft-lbs).

**Caution!**

It is absolutely necessary to tighten these bolts with a torque of 12 Nm (8.9 ft-lbs). Riding with loose bolts poses a high risk of accident and damage to the frame.

7. If applicable, proceed with the remaining steps for fitting the front boom, following the instructions given in the previous section "Removing and fitting the front boom".

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### Maintenance intervals

**Note:**

Please have all inspection and maintenance work performed by your authorized Hase retailer.

The first inspection should be done after the initial 300 km (200 miles). All additional inspections should be performed in 3000 km (2000 mile) intervals or on a yearly basis.

Component	Action	Before every ride	Once a month	Once a year
Lighting	Check function	X		
Tires	Check tire pressure Check tread wear and sidewalls	X X		
Brakes	Check pad thickness Check alignment to the brake disc Test braking effectiveness	X	X X	
Brake cables or brake hoses	Check for damage	X		
Dynamo mounting	Check for tightness	X		
Chain	Check for wear Oil		X X	
Crank bracket	Check bolts			X
Wheels	Check for lateral/radial true Check spoke tension		X X	
Handlebar	Inspect			X
Steering rod	Check for play			X
Headset	Check for play			X
Hubs	Check for play			X
Pedals	Check for axle play			X
Derailleur hanger	Check for damage		X	
Quick releases	Check for tightness	X		
Bolts and nuts	Check for tightness		X	
Bottom bracket assembly	Check for play			X

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Maximaler Luftdruck  
Maximum Pressure

### Checking wheels for lateral/radial true

The wheels have been carefully trued prior to shipment. During the break-in period, stress on the wheels may cause the spokes to settle into the rims so that the spoke tension is no longer uniform and the wheel untrue. Have your wheels re-trued by your authorized retailer at the first inspection after approx. 300 km (200 miles). Subsequent tensioning is seldom necessary. Check the spoke tension and lateral/radial true of the wheels regularly. You can check the spoke tension by squeezing together two adjacent spokes, working all the way around the rim. If you notice major differences in spoke tension, the wheel should be re-trued. You can check whether a rim is true by attaching a wire to the fork or fender stay and positioning it so that it almost touches the rim. Lift the Trix and rotate the wheel. If portions of the rim move away from or come in contact with the wire, the wheel must be taken to a qualified bicycle mechanic for re-truing.

#### **Note:**

The rear wheels do not have to be removed for the truing process.

### Checking tires

Check the tires for damage and tread wear. Tires with cuts, cracks, or worn tread should be replaced. Check tire pressure before each ride and always ride with the pressure specified on the tire sidewalls.

#### **Caution!**

Never ride with excessive tire pressure (higher than specified on the sidewall). The tire could burst. Risk of accident and injury!

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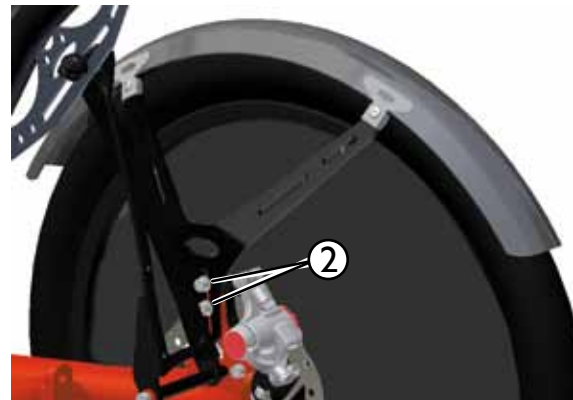
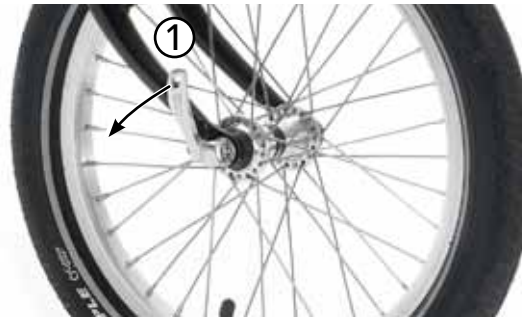
### Changing tires or repairing a flat

The front wheel of the Trix is mounted with a quick release. Simply release it to remove the front wheel. You do not have to remove the rear wheels for changing a tire or repairing a flat. The rear tires can be removed and remounted using a standard tire lever with the wheels attached. When removing and remounting tires, take care not to damage the fenders, and check that the rim tape is positioned correctly on the rim. The flexible spoke guards are clamped between the tire and the rim.

#### Note:

If you replace the original tire with a tire of a different size, it may be necessary to readjust the fenders and the position of the dynamo.

### Adjusting the fenders

In order to loosen the fender bolts, you must first adjust the seat angle so that the bolts are accessible.

1. Loosen the two Allen bolts (2) using a 5mm Allen key.
2. Slide the fender stay in the slot until it reaches the desired position, ensuring that the fender does not rub against the tire.
3. Tighten the two bolts (2) using a 5mm Allen key.

### Adjusting the dynamo (optional)

The dynamo is mounted on the left fender stay. To ensure the proper functioning of the dynamo, the dynamo roller must contact the wheel within the dynamo roller surface on the sidewall and the axis of roller rotation must be perpendicular to the roller surface. Please note the information provided in the user's manual from the dynamo manufacturer.

1. Loosen the bolt of the dynamo bracket so that the dynamo can be moved within the slot.
2. To assess proper alignment, hold a cord between the wheel-axis center and the dynamo-roller center, checking that the longitudinal axis of the dynamo (roller rotation axis) is located precisely along this line.
3. The dynamo roller must also be positioned so that it runs along the roller surface of the tire sidewall.

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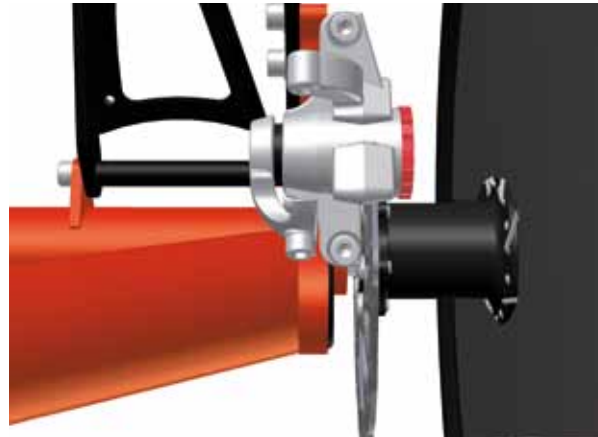
4. Retighten the bolt of the dynamo bracket.

**Caution!**

*Positioning the dynamo incorrectly will increase the running resistance and noise level and may cause damage to the tire sidewall or dynamo roller.*

**Checking disc brakes**

Please note the information provided in the enclosed manual from the brake manufacturer. Have a qualified bicycle mechanic check the pad thickness regularly. If you notice any unusual braking sounds or inadequate braking strength, consult your bicycle mechanic immediately.

**Checking and adjusting the internal gear**

Please note the information provided in the enclosed manual from the gear system manufacturer. The gear hub system has been carefully adjusted at the factory. Over time, however, the gear cables tend to stretch, and the gear hub may no longer function with the same precision. In this case, contact your authorized retailer.

**Note:**

*A complete readjustment of the gear hub system should be performed by a qualified mechanic. Incorrect adjustments can cause serious mechanical damage. Please note the information provided in the manual from the gear system manufacturer.*

**Caution!**

*After the gear hub system has been adjusted, it is absolutely critical that you test the gears in an open area away from traffic before taking the trike onto public roads. Problems may arise that could cause you to lose control of the trike.*

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### Checking the chain

The Trix's chain is longer than that of a standard upright bicycle. A long chain is generally subject to less wear. Therefore, it does not have to be changed as often as shorter chains. You can check the condition of the chain using a chain wear gauge from a bike shop. We recommend that you have a qualified bicycle mechanic check the condition of the chain and, if necessary, replace the chain, and possibly also the chainring.

The chain required for the longest frame adjustment of a Trix with hub gears and a quick-adjust frame is approx. 2430 mm (95 1/2"). This is equivalent to 96 chain links.

### Cleaning and oiling the chain

In order to ensure that the chain does not wear out before the end of its maximum working life, it must be cleaned and oiled on a regular basis.

1. Clean the chain using a clean, dry cloth.

#### **Caution!**

Never use solvents or pressure cleaners. These could wash the lubricants out of the chain bearings and shorten the life of the chain.

2. Apply chain oil, grease, or wax evenly to the chain links while slowly rotating the crank.

#### **Note:**

Rain can wash off traces of lubricant, releasing it into the environment. We recommend biodegradable lubricants.

#### **Caution!**

Never allow chain lubricant to come in contact with the brake discs or tires. This could significantly reduce braking performance or, in the case of lubricant on tires, wheel traction.

3. Continue rotating the crank to distribute the lubricant.
4. Wait several minutes to allow the lubricant to penetrate. Then wipe off all excess lubricant using a clean, dry cloth.

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### **Cleaning and disinfecting your Trix**

Clean your Trix regularly. Dirt, as well as salt from winter roads, can damage the trike. Clean the trike using a sponge and soapy water (ideally with a biodegradable household cleaning product). You can also clean your trike using a gentle stream of water. However, do not use a pressure washer. The pressurized water can penetrate bearings and damage them. After cleaning the trike, clean and lubricate the chain. The handlebar grips and seat can be cleaned and disinfected using standard cleaning agents and disinfectants. After cleaning the trike, clean and lubricate the chain. Wipe all surfaces with a moist, lint-free cloth.

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. Only use disinfectants that do not damage plastic parts, e.g. aldehyde-based disinfectants.

### **Checking the chain tube**

The purpose of the chain tube (1) is to protect your clothing from chain lubricant. Over time, friction from the moving chain will cause the tube to wear. Check the ends of the chain tube for any signs of wear. If the ends are completely worn, have your bicycle mechanic replace the chain tube.

**Note:**

In order to replace the chain tube, the chain must be opened at the connecting link.

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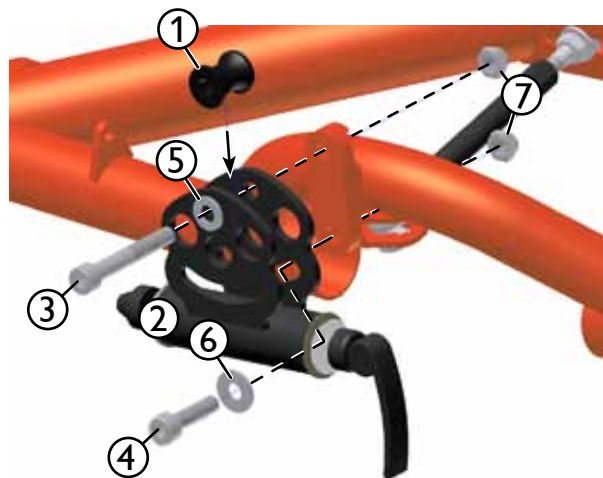
With the universal coupling, the Trix can be used in combination with a second Trix as a tandem trike. You can also attach a BOB trailer, a Weber Monoporter, or a trailer with a straight or curved drawbar and a Weber coupling. If you live in Germany, however, keep in mind that, according to German road traffic regulations (StVZO), only riders who are at least 16 years of age are allowed to pull a bike trailer. The Trix can also be pulled as a trailer behind a Kettwiesel. The picture on the left shows all parts included in the coupling assembly. However, depending on the application, not all parts will be required.

Put the aluminum spacer (1) into the hitch (2). Attach the hitch (2) to the Trix frame using the longer M8 bolt (3) with the small washer (5) and self-locking nut (7), positioning it with the slot beneath the two round holes. Insert the shorter M8 bolt (4) with the washer (6) through the slotted hole and fix it with the self-locking nut (7). The required tightness of the bolts (torque) depends on the intended use of the hitch.

The additional fittings required will depend on the intended use.

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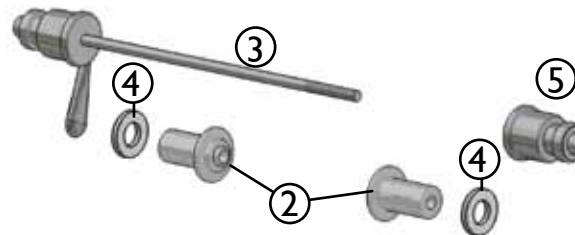
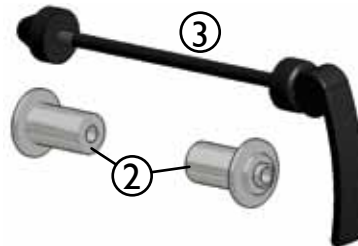
### For use with a second Trix in tandem

If you are using the hitch to attach a second Trix in tandem, the two M8 bolts with nuts (1) and (4) must be loose enough to allow the hitch to pivot freely from the top bolt.

**Caution!**

The hitch must be able to pivot freely, and the contact surfaces, well greased; otherwise, when cornering, the fork of the second Trix could be wrenched out of the coupling and damaged.

1. Insert the long ends of the spacers (2) into the barrel of the coupling.
2. Detach the front wheel of the second Trix and remove the quick release (3).
3. Slide the fork onto the spacers of the coupling (2).
4. Insert the quick release (3) of the second Trix through the coupling and secure it tightly.

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2. Insert the short ends of the spacers (2) into the barrel of the coupling.
3. Remove the two centering springs from the quick-release axle of the BOB trailer.
4. Insert the BOB quick release in the coupling with an M6 washer (4). Put a second M6 washer (4) onto the other side of the quick release and screw on the quick-release nut (5).
5. Slide the BOB trailer onto the coupling and tighten the quick release.

**Note:**

Please note the information provided in the user's manual for the BOB trailer.

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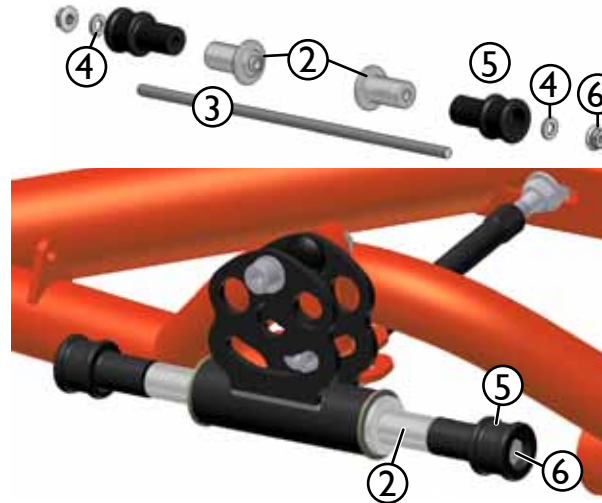
### For use with a Weber Monoporter

1. Position the hitch so that it is level and screw it to the attachment plate of the Trix frame with a torque of 30 Nm (22 ft-lbs).

**Caution!**

The hitch must be screwed on tightly.

2. Insert the short ends of the spacers (2) into the barrel of the coupling.
3. Insert the threaded rod (3) through the coupling.
4. Place a brass tube with the plastic adaptor (5) from the Weber trailer over each end. These parts are not included in this coupling assembly.
5. Place the two small M6 washers (4) over the ends of the threaded rod and secure them with an M6 nut (6) on each end.
6. Slide the Monoporter onto the coupling.

**Note:**

Please note the information provided in the user's manual for the Monoporter.

### For use with a Weber coupling and straight drawbar

1. Position the hitch so that it is level and screw it to the attachment plate of the Trix frame with a torque of 30 Nm (22 ft-lbs).

**Caution!**

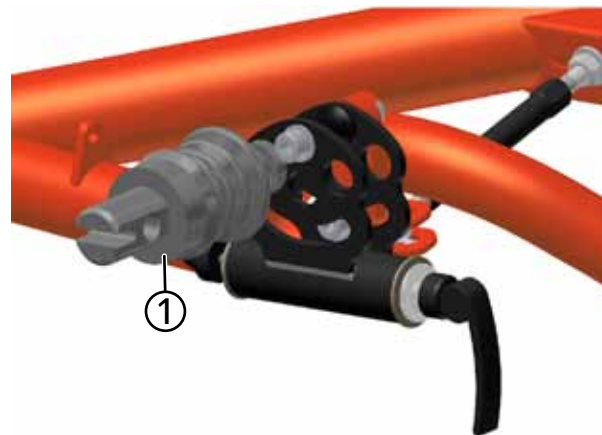
The hitch must be screwed on tightly.

2. Remove the connecting plate of a commercially available Weber coupling type E or EL (not included in the universal coupling assembly).
3. Insert the Weber coupling (1) into the middle hole of the universal coupling and attach it in the same way it was attached to the connecting plate. Make sure that the projections on the plastic washer are positioned in the groove of the Weber coupling.

**Caution!**

This coupling is only appropriate for trailers with straight drawbars. When pulling a trailer with a curved drawbar, the turning circle for left turns is severely limited because the curved drawbar bumps against the left rear wheel of the Trix. Danger of accident! For curved drawbars, only use the adapter (2) that has been specially designed for this purpose.

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### For use with a Weber coupling and curved drawbar

In order to keep the turning circle for left turns small, we recommend using this adapter for the attachment of (standard) curved drawbars.

This adapter can be used separately or in connection with the universal coupling.

1. Place the Weber coupling adapter on the attachment plate of the Trix frame and attach it by means of the two M8 bolts and self-locking nuts with a torque of 30 Nm (22 ft-lbs).
2. Insert the Weber coupling (1) into the hole of the adapter (2) and attach it in the same way it was attached to the connecting plate. Make sure that the projections on the plastic washer are positioned in the groove of the Weber coupling.

**Note:**

The adapter can also be used together with the universal coupling. In this case, you must use longer M8 bolts.

**Caution!**

The hitch must be screwed on tightly.

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### Attaching the headrest

The headrest is mounted in the tubes of the seat back. The assembly includes a headrest frame with a pad and two inserts (1) that fit into the ends of the seat-back tubes.

To attach the headrest, you will need the following tools: a utility knife, a lighter, and possibly a rubber mallet.

1. Insert the utility knife through the seat-cover material covering the holes and carefully cut around the inner edge of the tubes, making openings large enough to accommodate the inserts (1).
2. Carefully melt and seal the cut edges of material using the lighter.
3. Push the inserts (1) into the ends of the seat-back tubes, making sure they are fully inserted. Use the rubber mallet if necessary. If you have aluminum tubes, insert at first the adapter (8).
4. Screw the two mounting brackets (7) onto the inserts (1).
5. Place the clamping rings (3) over the clamping bolts (2) and insert the bolts through the slots of the mounting brackets (7). Place washers (4) over the bolts and screw on the cap nuts (5) without tightening.
6. Insert the tubes of the headrest frame (6) through the holes in the clamping bolts (2) and screw the cap nuts (5) without tightening.
7. The height of the headrest can be adjusted by sliding the frame in the clamping bolts.
8. The distance between the seat and the headrest can be adjusted by folding the mounting brackets inward or outward while sliding the clamping bolts along the slots.
9. When the headrest is in the desired position, tighten all bolts.

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## Long-term storage and disposal

### Preparing the Trix for long-term storage, and for use after long-term storage

If you plan to store the Trix for a longer period of time, e.g. during the winter months, then prepare the trike for storage as follows:

1. Clean the Trix as described in the chapter “Cleaning your Trix” on page 26.
2. Lubricate or wax the chain.
3. Tires gradually lose air when in storage. Trikes stored on flat tires for an extended period may damage the tires’ sidewalls. Therefore, it is advisable to hang the Trix so that the tires are not in contact with the floor. Otherwise, be sure to check the tire pressure of the stored trike regularly.
4. Store the trike in the lowest gear. This ensures that the gear cable is under minimal tension.
5. Store the Trix in a dry and dust-free room. It may be wise to cover the trike with a tarp or similar dust cover.
6. Before using a Trix that has been in storage, make sure that the trike is in perfect working order and have your authorized retailer perform a maintenance check.

### Preparing the Trix for disposal

The construction of the Trix is extremely durable and robust. However, if the trike must be disposed of, e.g. owing to irreparable damage following an accident, then contact your authorized retailer, who will help you dispose of the Trix, or individual parts, in an environmentally sound manner. Furthermore, be sure to dispose of any parts that you replace – such as tires, chains, brake pads, etc. – in an environmentally acceptable manner. In case of doubt, contact your authorized retailer.

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## Conditions of warranty and liability

The terms and conditions are based on the statutory warranty. This excludes parts subject to wear, such as the chain, tires, brakes, etc. The warranty period (statutory) of 24 months commences on the date of purchase. In addition, Hase offers a 3-year warranty (from date of purchase) against breakage of the frame in accordance with the following conditions. By sending in the enclosed registration card, you can extend the warranty period to 5 years. Your dealer is obliged to ensure, among other things, that your bicycle has no defects that diminish its value or suitability for the intended use. The exact details will vary according to your country. In Germany, this liability ends two years after purchase. Your dealer must assemble and adjust your bicycle to ensure safe function. The dealer also has to perform a final safety check and a test ride.

- Hase will only be held liable for defects resulting from faulty frame manufacturing and assembly. For assembly parts, the relevant manufacturer is liable.
- This additional warranty only applies to the original owner and to usage in accordance with the user's manual.
- Within the warranty period of 3 years, Hase will repair, or if necessary replace, any broken frame part free of charge. Any defective parts replaced by Hase become the property of Hase.
- Any further claims beyond the scope of this warranty are excluded. Hase will not cover the costs of assembly or disassembly (e.g., by the retailer) or shipping. We will not accept postal deliveries with insufficient postage.
- The maximum permissible load (rider and luggage) of 120kg (265 lbs) specified for the Trix must not be exceeded.
- If the Trix is used in competition, ridden on rough terrain, or subjected to abnormally excessive stress, the statutory and additional warranty becomes void.
- Damage caused by accidents is not covered by the warranty.
- The warranty becomes void if the Trix has been fitted with incompatible accessories or if accessories are installed incorrectly. Therefore always have components replaced and installed by an authorized retailer.
- The warranty becomes void if improper modifications or alterations (e.g., grinding, drilling, bending, etc.) are made to the frame.
- It is not possible to extend the warranty or commence a new warranty period.
- Warranty claims can only be handled by the authorized Hase retailer from whom the trike was purchased or by one of our distribution partners.
- The Trix may only be ridden by children under parental supervision.